

CERAM-ETCH™

Porcelain Etching Gel

*Ceram-Etch is a 9.5% hydrofluoric acid etching gel used for etching porcelain. Ceram-Etch will **not** etch metal. For professional use only.*

Caution! Contains Acid. See SDS for precautions.

Directions: At room temperature, apply Ceram-Etch to areas of porcelain where an etch is desired. For best results, etch regular feldspathic porcelain for 5 minutes, pressable ceramics for 1 minute, and lithium ceramics for 20 secs. After etching, rinse with water to remove the gel. Place the restoration in a beaker of distilled water and clean ultrasonically for 2 minutes. When dry, the etched surface should look frosted. Do not remove this surface. The restoration is now ready for the silane and composite.

Gresco Products, Inc.
13391 Murphy Road ♦ Stafford, TX 77477
(800) 527-3250 ♦ (281) 261-1811

www.grescoproducts.com

Rev: Dec 2016