

# CERAM-ETCH™

*Porcelain Etching Gel*

*Ceram-Etch is a 9.5% hydrofluoric acid etching gel used for etching dental porcelain. Ceram-Etch will **NOT** etch metal or zirconia. For professional use only.*

***Caution! Contains Acid. See SDS for precautions.***

**Directions:** At room temperature, apply Ceram-Etch to areas of porcelain where an etch is desired. For best results, etch regular feldspathic porcelain for 5 minutes, pressable ceramics for 1 minute, and lithium disilicate ceramics for 20 secs. After etching, rinse with water to remove the gel. Place the restoration in a beaker of distilled water and clean ultrasonically for 2 minutes. When dry, the etched surface should look frosted. Do not remove this surface. The restoration is now ready for the silane and composite.

*Gresco Products, Inc.*  
13391 Murphy Road ♦ Stafford, TX 77477  
(800) 527-3250 ♦ (281) 261-1811

**[www.grescoproducts.com](http://www.grescoproducts.com)**

*Rev: Jan 2023*