

CERAM-ETCHTM PC

Pressable Ceramic Etching Gel

*Ceram-Etch PC is a 5% hydrofluoric acid etching gel used for etching dental pressable ceramics such as e.max and Empress. Ceram-Etch will **not** etch metal. For professional use only.*

Caution! Contains Acid. See MSDS for precautions.

Directions: Apply Ceram-Etch PC to areas of the ceramic where an etch is desired. For best results, etch Empress and other lucite pressable ceramics for 1 minute at room temperature. Etch e.max and other lithium disilicate ceramics for 20 seconds at room temperature. After etching, rinse with water to remove the gel. Place the restoration in a beaker of distilled water and clean ultrasonically for 2 minutes. When dry, the etched surface should look frosted. Do not remove this surface. Do not over etch. Note: Use Ceram-Etch 9.5% when etching regular feldspathic porcelains.

Gresco Products, Inc.

13391 Murphy Road ♦ Stafford, TX 77477

(800) 527-3250 ♦ (281) 261-1811

www.grescoproducts.com

Rev Jan 2023